



Heal Naturally.
Impress Easily.



Encode[®] Emergence

3Shape Lab Manual



Traditional and Stone Model Impressions

Traditional Impression

Inspect the case for an accurate impression of the Encode Emergence Healing Abutment. Impressions of the coded occlusal surface must be clearly visible. The total circumference should be clean and above the soft tissue; codes should be visible with no rips, tears, bubbles or distortion.

Stone Model

Use low expansion die stone, for proper scanning, the case must provide the following:

1. A defect free occlusal surface and clear view of the surrounding tissue
2. No voids or bubbles in the stone

Weigh and measure die stone and water per the manufacturer's instructions. Allow the stone to fully set. Do not flip or mount casts prematurely.


Mount the casts on an articulator compatible with Adesso Split Mounting plates using the bite registration.

Casts must be centered on mounting plates; the scanner reads where the plates are located. If the cast is off the plate, the case will not be scanned correctly.


Customized Abutment Order Creation In 3Shape

IMPORTANT: Prior to creating an order and designing the final abutment in 3Shape, look at the stone model to determine the implant type; Implant connection is indicated by the number of grooves on the occlusal portion of the screw, 2 grooves for Certain and 3 grooves for TSV implant connection.


1. In the 'Scan settings' section of the menu, for 'Object type' select "Model". If antagonist is available, under 'Antagonist', select applicable options

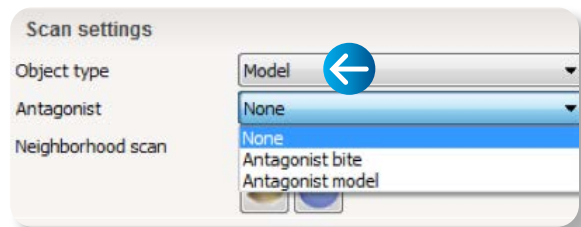

2. Select the "Abutment" icon in the menu (customized abutment is selected by default)
3. Abutment Order Settings
 - a. For 'Category', select "Zimmer Biomet TSV" or "BIOMET 3i"
 - b. For 'System', select "ZB (TSV or Certain) EHA"

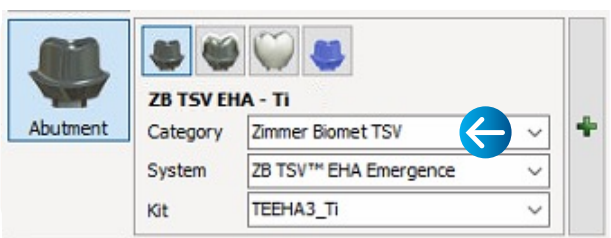

- c. For 'Kit', select the correct Encode Healing Abutment catalog code based on the code scheme. If necessary, measure the Emergence Profile to the correct diameter. Refer to ZVINST0103 for Encode Emergence Code Scheme Matrix

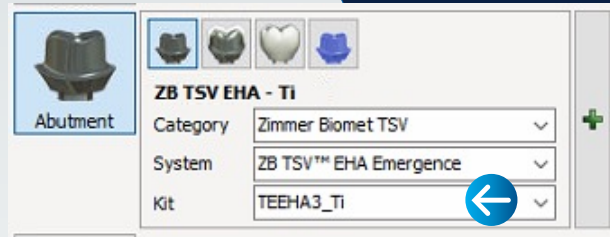


Customized Abutment Scanning

1. Insert the stone model with Encode Healing Abutment into the 3Shape scanner and scan
2. Scan antagonist/ bite registration, if available
3. Start the overview scan
4. Start detailed area scan
5. Start the alignment process
 - a. To align an abutment click on "Alignment Setting"
 - b. In the scanned EHA, a 1-point alignment may be sufficient to align Encode Emergence healing abutment. Use any area on the occlusal dome of the abutment
 - For the 3.8mm emergence profile is recommended the use of "3-point alignment" option
 - Optional: Select box for "3-point alignment"


Do not use the dimples on the occlusal as alignment points

- c. In the virtual mode, select and position the dot(s) in the same location as in the step above
- d. The result of the alignment is displayed on the screen
6. Select "Difference map" or "2D Cross" to confirm accuracy of alignment:
7. Complete the scanning


Example of Difference Map - Successful Alignment:

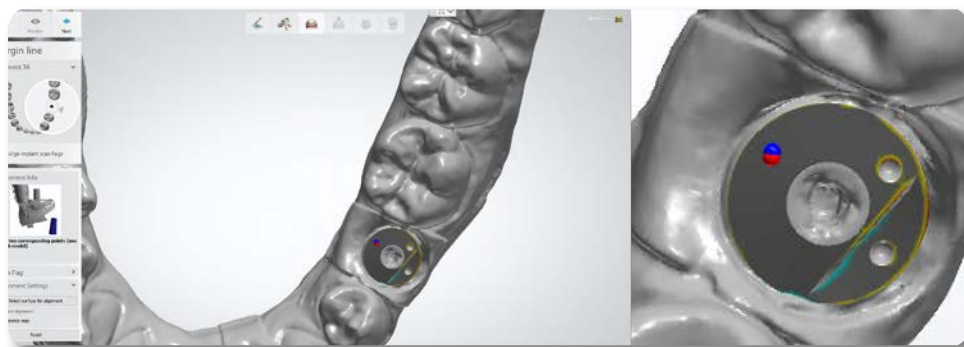


Example of Difference Map - Unsuccessful Alignment:

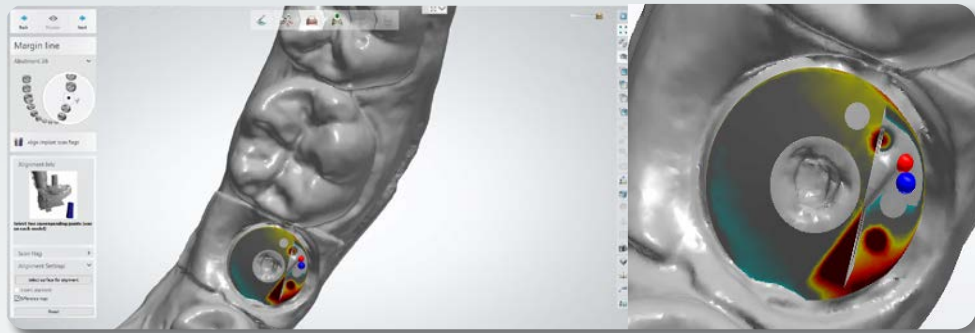

Example of 2D Cross - Successful Alignment:

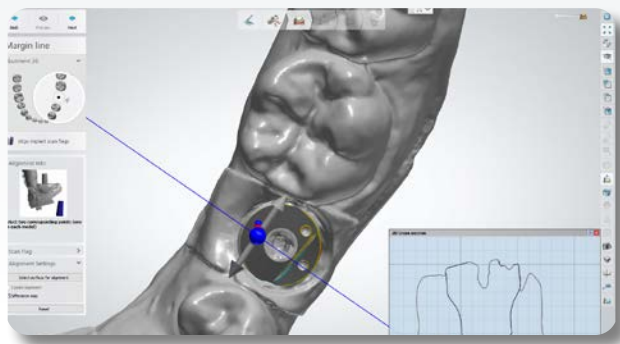


Example of 2D Cross - Unsuccessful Alignment:

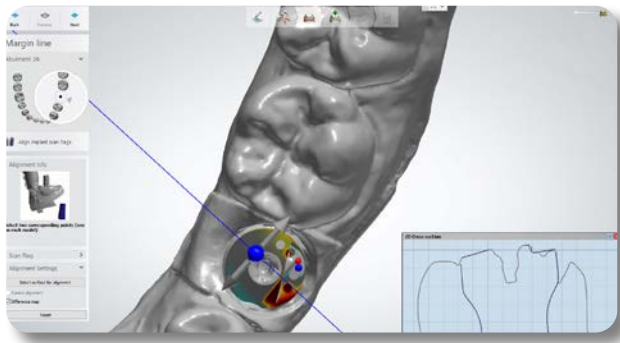

Abutment Design

1. Proceed with standard abutment design
2. Click "Next"

NOTE: If a Zirconia abutment is being designed, "Use screw hole" must be selected at all times.

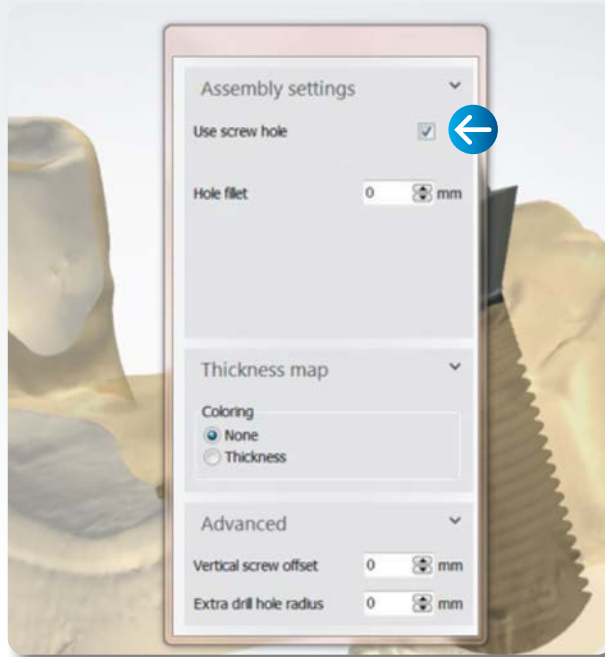

NOTE: For cases requiring a model, please refer to 3Shape Modelbuilder instructions.

The 'Analog to Model Spacing' value defaults to "0.020 mm". If a tighter or looser fit of the model analog is required, then adjust the 'Analog to Model Spacing' value. The 'Protect Analog' option defaults to "True". The 'Protect Analog Distance' value defaults to "1.000 mm" and can be adjusted as desired.

Submit Case Through Dental Manager


1. Send to manufacturer from your outbox to Zimmer Biomet Dental inbox

Intra-Oral Scanner Impression (Non-TRIOS)

Customized Abutment Order Creation In 3Shape

IMPORTANT: Prior to creating an order and designing the final abutment in 3Shape, look at the .STL file to determine the implant type; Implant connection is indicated by the number of grooves on the occlusal portion of the screw, 2 grooves for Certain and 3 grooves for TSV implant connection.

1. In the 'Scan settings' section of the menu, for 'Object type' select "Digital impression". If antagonist is available, under 'Antagonist', select applicable options

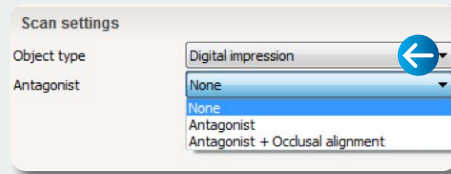

NOTE: "gingival scan" option is selected by default; this option should always be unselected when doing an IOS Impression case.

Incorrect



Correct




2. Select the "Abutment" icon in the menu ("Customized abutment" is selected by default)

3. Abutment Order Settings


- a. For 'Category', select "Zimmer Biomet TSV" or "BIOMET 3i"



- b. For 'System', select "ZB (TSV or Certain) EHA Emergence"

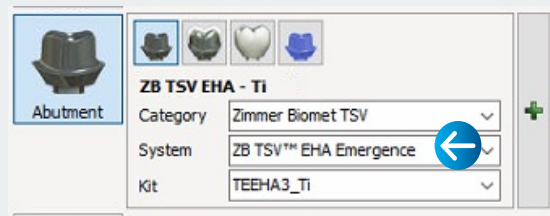


- c. For 'Kit', select the correct Encode Healing Abutment catalog code based on the code scheme. If necessary, measure the Emergence Profile to the correct diameter. Refer to ZVINST0003 for Encode Emergence Code Scheme Matrix

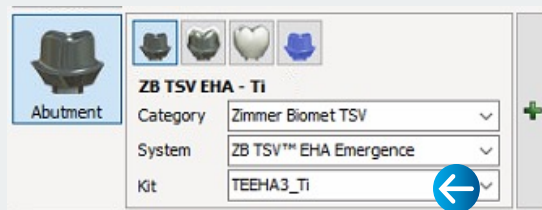

4. For cases requiring a 3D-printed model, click on "Model" icon in menu

Importing Intra-Oral Scan

1. After order is created, in the 'Orders' section, right click on the order that has been created and select "Import scan" to load the .STL file. Order will appear with yellow exclamation point "!"

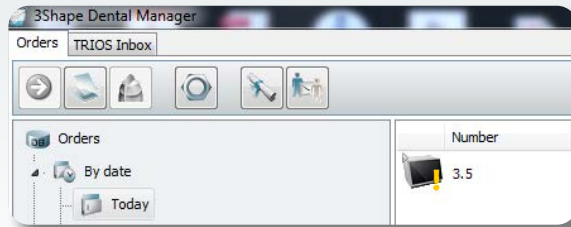


2. Scan antagonist/ bite registration, if available
3. Start the alignment process
 - a. To align an abutment click on "Alignment Setting"
 - b. In the scanned EHA, align the dot/position on any area on the occlusal dome of the abutment
 - For the 3.8mm emergence profile is recommended the use of "3-point alignment" option
 - Optional: Select box for "3-point alignment"


Do not use the dimples on the occlusal as alignment points

- c. In the virtual mode, select and position the dots in the same location as in the step above
 - d. The result of the alignment is displayed on the screen
4. Select "Difference map" or "2D Cross" to confirm accuracy of alignment:


Example of Difference Map - Successful Alignment:

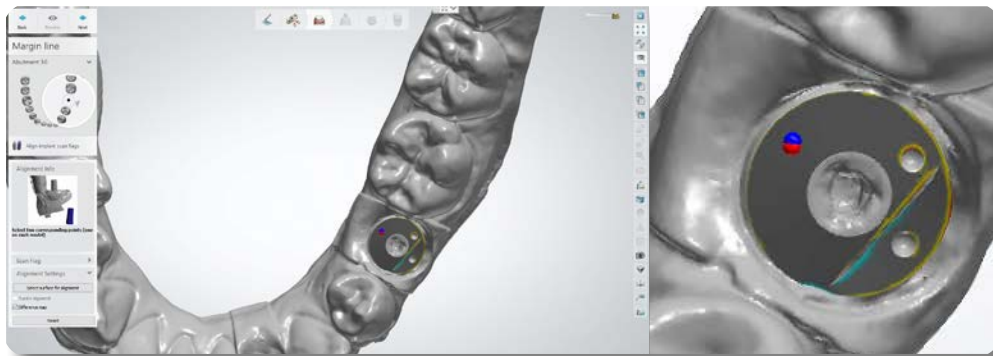


Example of Difference Map - Unsuccessful Alignment:

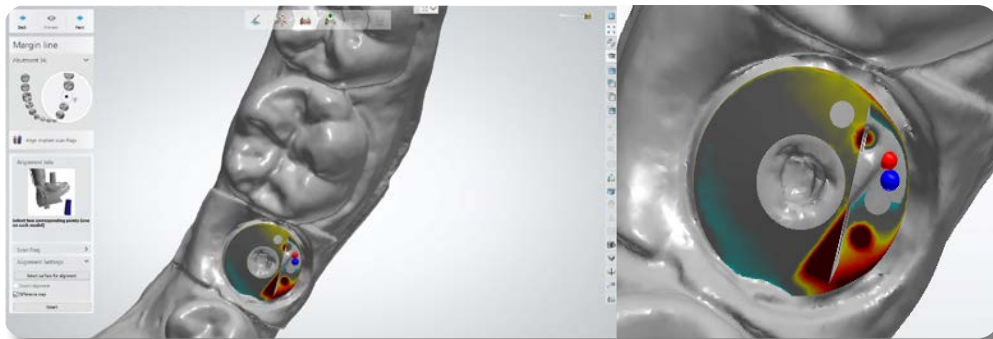

Example of 2D Cross - Successful Alignment:

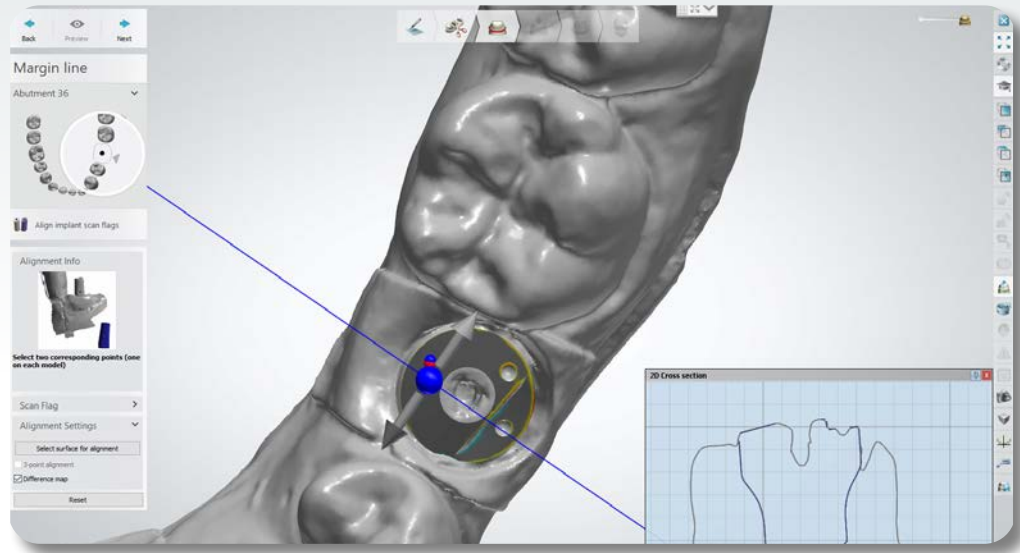

Example of 2D Cross - Unsuccessful Alignment:

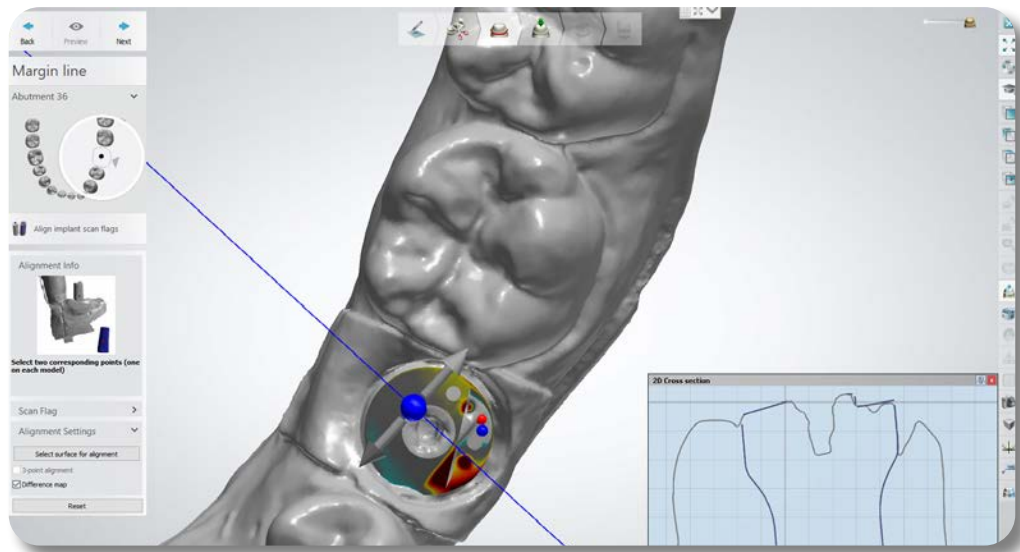


7. Complete the creation of the order

*per 3Shape instructions

Abutment Design

1. Proceed with standard abutment design
2. Click "Next"

NOTE: If a Zirconia abutment is being designed, "Use screw hole" must be selected at all times.

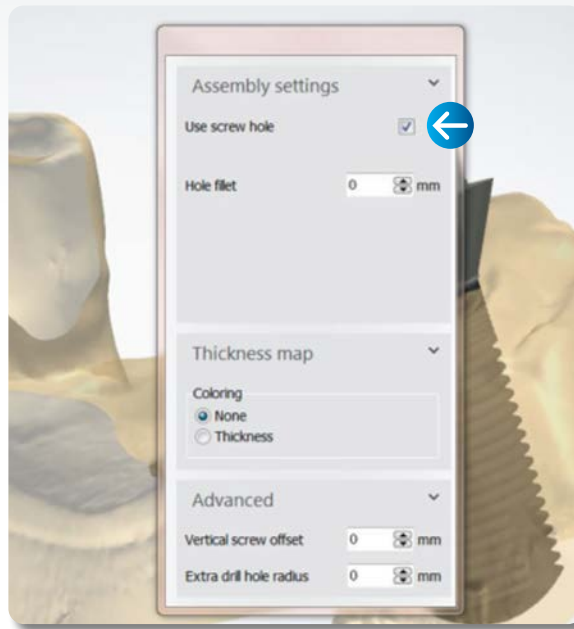

NOTE: For cases requiring a model, please refer to 3Shape Modelbuilder instructions.

The 'Analog to Model Spacing' value defaults to "0.020 mm". If a tighter or looser fit of the model analog is required, then adjust the 'Analog to Model Spacing' value. The 'Protect Analog' option defaults to "True". The 'Protect Analog Distance' value defaults to "1.000 mm" and can be adjusted as desired.



Submit Case Through Dental Manager

1. Send to manufacturer from your outbox to Zimmer Biomet Dental inbox


Intra-Oral Scanner Impression (TRIOS®)

Importing TRIOS Scan*

1. Open the 'TRIOS Inbox' tab, right click on the applicable case and select "Accept"
2. Select "Accept" for the following message in order to move the case into the 'Orders' tab and start order creation

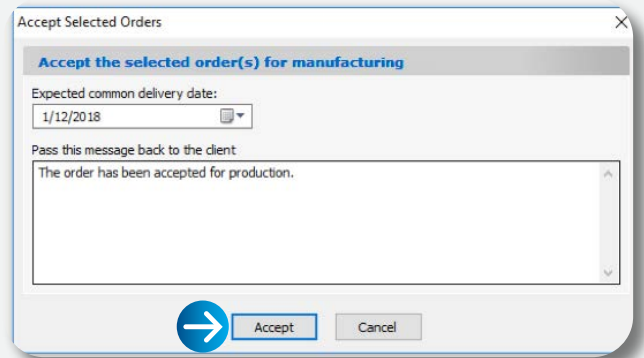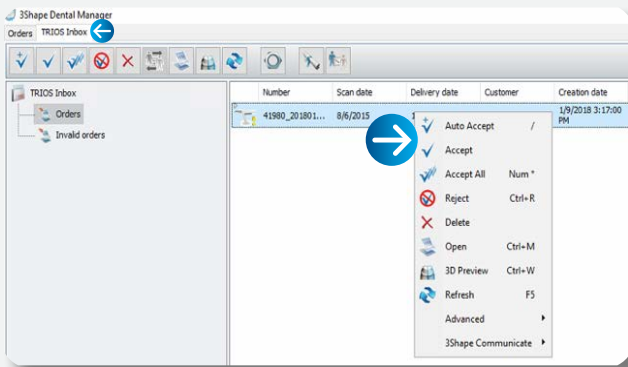


3. Once the case is in the 'Orders' tab, right click on the case and select "Modify"

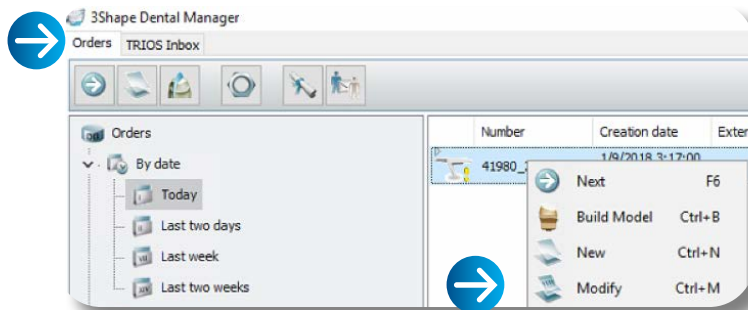


Customized Abutment Order Creation In 3Shape

IMPORTANT: Prior to modifying the order and selecting the correct kit, select "3D Preview" to view the scan and determine the implant type; Implant connection is indicated by the number of grooves on the occlusal portion of the screw, 2 grooves for Certain and 3 grooves for TSV implant connection.

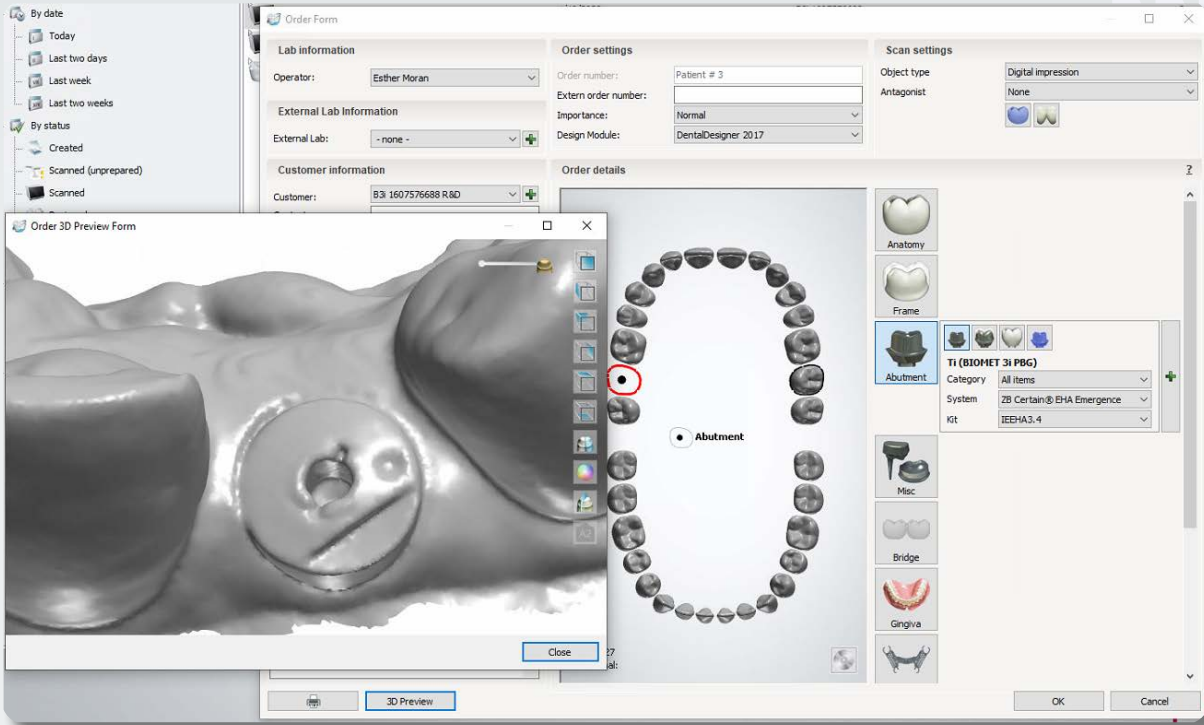

1. In the 'Scan settings' section of the menu, for 'Object type' select "Digital impression". If antagonist is available, under 'Antagonist', select applicable options

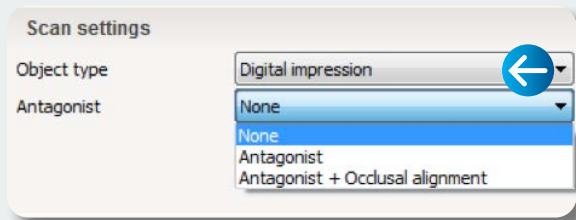

NOTE: "gingival scan" option is selected by default; this option should always be unselected when doing an IOS Impression case.

Incorrect




Correct




2. Select the "Abutment" icon in the menu (customized abutment is selected by default)

3. Abutment Order Settings


- a. For 'Category', select "Zimmer Biomet TSV" or "BIOMET 3i"

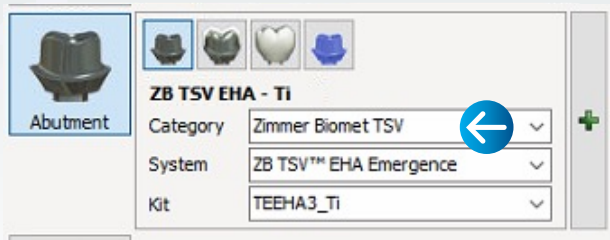

- b. For 'System', select "ZB (TSV or Certain) EHA Emergence"

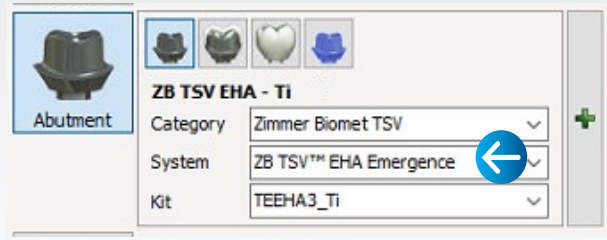

- c. For 'Kit', select the correct Encode Healing Abutment catalog code based on the code scheme. If necessary, measure the Emergence Profile to the correct diameter. Refer to ZVINST0003 for Encode Emergence Code Scheme Matrix

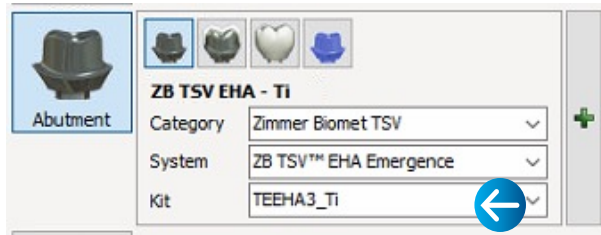


4. For cases requiring a 3D-printed model, click on the "Model" icon in menu
5. Once order has been modified click "Next" to begin alignment process
6. Start the alignment process
 - a. To align an abutment click on "Alignment Setting"
 - b. In the scanned EHA, align the dot/position on any area on the occlusal dome of the abutment
 - For the 3.8mm emergence profile is recommended the use of "3-point alignment" option
 - Optional: Select box for "3-point alignment"


Do not use the dimples on the occlusal as alignment points

- c. In the virtual mode, select and position the dots in the same location as in the step above
- d. The result of the alignment is displayed on the screen
7. Select "Difference map" or "2D Cross" to confirm accuracy of alignment:


Example of Difference Map - Successful Alignment:

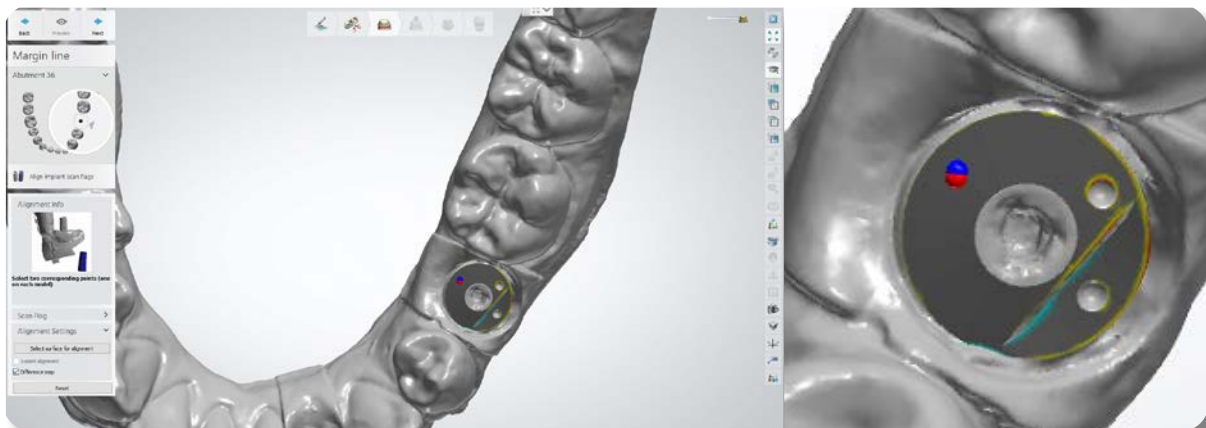


Example of Difference Map - Unsuccessful Alignment:

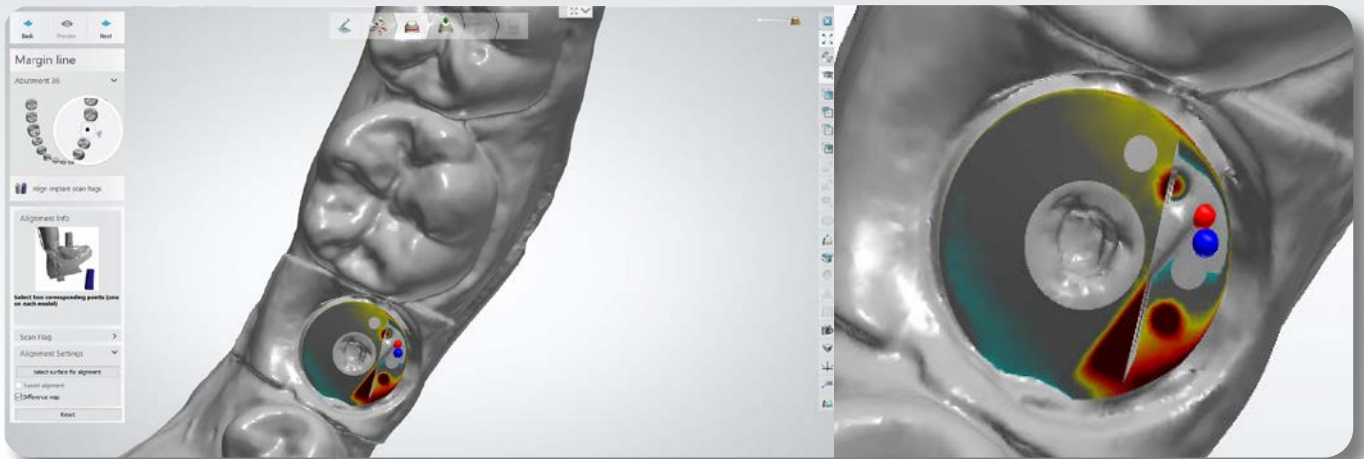

Example of 2D Cross - Successful Alignment:

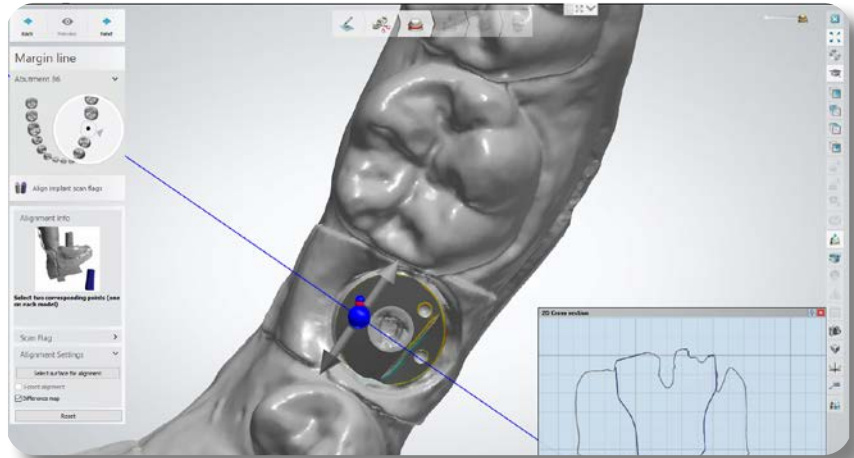

Example of 2D Cross - Unsuccessful Alignment:

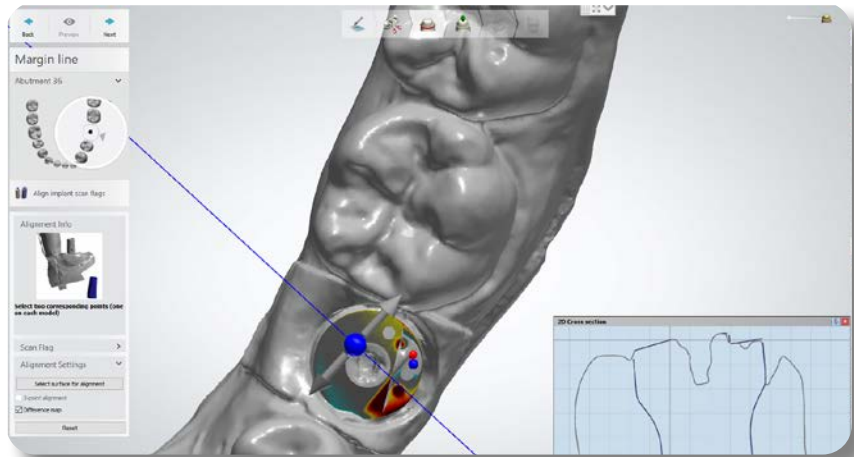


8. Complete the creation of the order

Abutment Design

1. Proceed with standard abutment design

NOTE: If a Zirconia abutment is being designed, "Use screw hole" must be selected at all times.

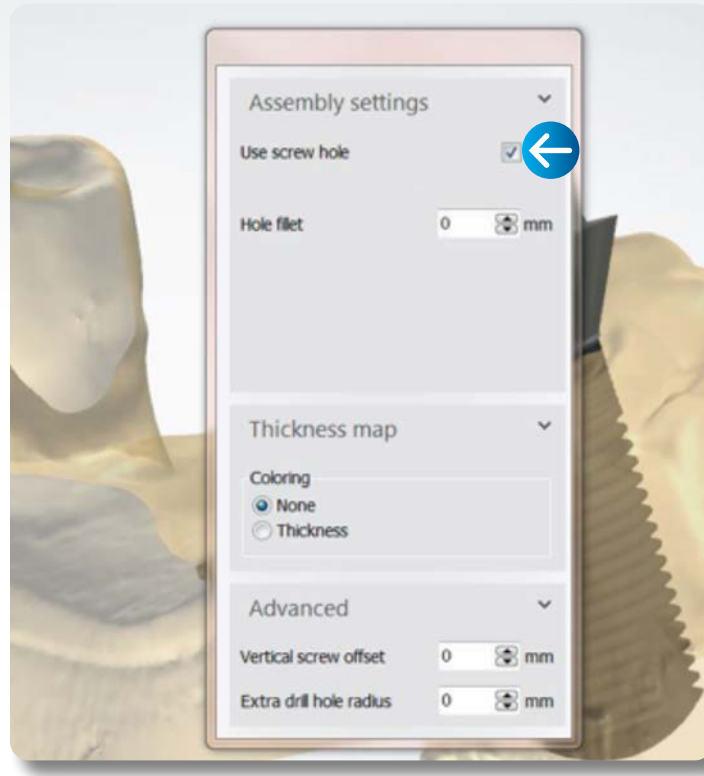

NOTE: For cases requiring a model, please refer to 3Shape Modelbuilder instructions.

The 'Analog to Model Spacing' value defaults to "0.020 mm". If a tighter or looser fit of the model analog is required, then adjust the 'Analog to Model Spacing' value. The 'Protect Analog' option defaults to "True". The 'Protect Analog Distance' value defaults to "1.000 mm" and can be adjusted as desired.

2. Click "Next"

Submit Case Through Dental Manager

1. Send to manufacturer from your outbox to Zimmer Biomet Dental inbox



ZimVie Dental
4555 Riverside Drive
Palm Beach Gardens, FL 33410
Tel: +1-561-776-6700
Fax: +1-561-776-1272

Unless otherwise indicated, as referenced herein, all trademarks and intellectual property rights are the property of ZimVie Inc. or an affiliate; and all products are manufactured by one or more of the dental subsidiaries of ZimVie Inc. (Biomet 3i, LLC, Zimmer Dental, Inc., etc.) and marketed and distributed by ZimVie Dental and its authorized marketing partners. For additional product information, please refer to the individual product labeling or instructions for use. Product clearance and availability may be limited to certain countries/regions. This material is intended for clinicians only and does not comprise medical advice or recommendations. Distribution to any other recipient is prohibited.

This material may not be copied or reprinted without the express written consent of ZimVie.

ZVINST0003 REV A 05/22 ©2022 ZimVie.

All rights reserved.

